

## ABOUT US

Founded in 2015, we are a proud American company, based in the heart of Los Angeles, California, that is deeply committed to improving the lives of seniors and low vision individuals.

Our team of passionate individuals is driven by a strong sense of purpose, and we are dedicated to putting a smile on the faces of those who have struggled to read. We believe that everyone deserves the simple pleasure of reading, and it is this belief that motivates us to work tirelessly towards achieving our mission.

## NEED HELP?

We invest heavily on product research and development to ensure **Magnipros** products exceed our customers expectations.

If you are not completely satisfied with your purchase, please contact us so that we can quickly resolve the issue - and please provide your order number for the most efficient service.


---


**WE QUICKLY RESPOND TO EMAILS WITHIN 24 HOURS**

---


### CONTACT US:

 Email: [support@magnipros.com](mailto:support@magnipros.com)

 Toll Free: **1.888.507.8962**

 Message us on Facebook.

### SUPPORT HOURS:

 **Monday-Friday; 8:30am-5pm PST.**



# LED PAGE MAGNIFIER


(Item #1820+ST)

# THANK YOU FOR YOUR PURCHASE

REGISTER YOUR PRODUCT  
AND RECEIVE A  
**BONUS YEAR WARRANTY**

## HOW TO USE PAGE MAGNIFIER

PLEASE READ THIS INSTRUCTION MANUAL CAREFULLY BEFORE USE

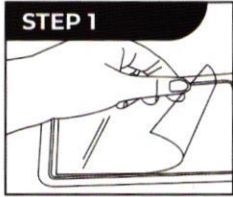

### STEP 1

Please remove the protective film on the flatter side of the lens before using this magnifier.


*Note: One protective film only.*



### STEP 2


Follow polarity indicators and insert 3 AAA batteries (not included) into battery compartment.

*\*Tip: Install batteries on both sides first, then install the last battery in the middle position of battery compartment.*

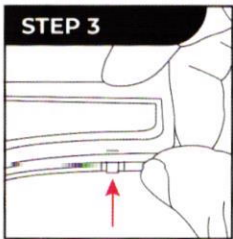

### STEP 3


Tap the switch/button once to turn on or change lighting sources, long press/hold the button to increase or decrease brightness level to your desired lighting.

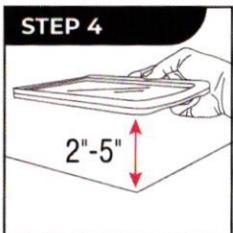

### STEP 4

Please hold this led page magnifier 2"-5" above objects intended to magnify for optimal magnifying result.

## HOW TO USE HANDS-FREE STAND

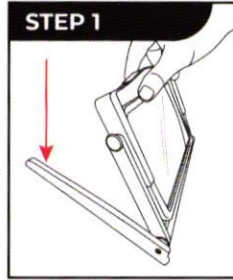


### STEP 1

Find a flat surface like a desk or a table where you want to use your page magnifier and unfold the stand demonstrated in the diagram.




### STEP 2

Fine-tune the stand's positioning to achieve the optimal angle and spot that suits your particular requirements.

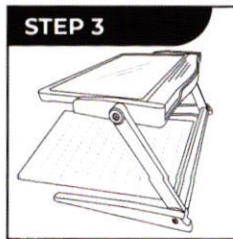

### STEP 3

Put the document, book, or object you want to magnify under the page magnifier. Adjust the stand and magnifier as needed to get a clear, magnified view.

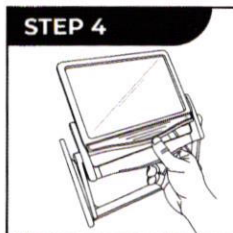

### STEP 4

**To Detach the magnifier from the stand:** Apply mild outward pressure to the magnifier and carefully slide it out shown in the diagram.

# ACTIVATE YOUR EXTENDED WARRANTY TO 2 YEARS

Register within 10 days of receiving this magnifier at [magnipros.com/pages/product-registration](https://magnipros.com/pages/product-registration)

or

Use your phone camera to scan this QR code, you will be completing on messenger within 2 mins.



#MagniPros